

# Wearing Your 3MAura™

Remove the respirator from its packaging and hold with straps facing upward. Place the bottom strap under the center flaps next to the "ATTENTION" statement.

Fully open the top and bottom panels, bending the nosepiece around your thumb at center of the foam. Straps should separate when panels are opened. Make certain the bottom panel is unfolded and completely opened.

Place the respirator on your face so that the foam rests on your nose and the bottom panel is securely under your chin.

Adjust for a comfortable fit by pulling the top panel toward the bridge of your nose and the bottom panel under your chin.

Make certain hair, facial hair, jewelry and clothing are not between your face and the respirator as they will interfere with fit.

Store or discard according to your facility's infection control policy.